

Department of Public Safety - Alcohol Testing

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)


ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST R. ROSADO #557 AGENCY LA PAZ COUNTY SO
DATE 01-23-15 08:08 HOURS TIME
INTOXILYZER SERIAL # ~~80-001197~~ 80-001197 LOCATION LPC SO COUNTY PARK SUBSTATION

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 101 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  #557

DPS Form Exh G-2 (Rev.05-01)

Historical Note

New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000

Location: BACK UP/COUNTY PARK

Serial Number: 80-001197

Core Version: 8105.48

01/23/2015

08:08:31

Standard Lot#: 14014100A1

Last Changed By: R. ROSADO #557

QAS: R. ROSADO #557

LA PAZ CSO

Operator: R. ROSADO #557

LA PAZ CSO

Test g/210L Time

Air Blank 0.000 08:09:01
0.100 Cal Check 0.101 08:09:22
Air Blank 0.000 08:09:51

Preventative Maintenance
Performed -- Timer Reset