


Department of Public Safety - Alcohol Testing

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST R. Rosado #557 AGENCY LA PAZ COUNTY SO  
DATE 01-23-15 10:52 HOURS TIME  
INTOXILYZER SERIAL # 80-001198 LAPAZSO LOCATION LPCSO EHRIGBERG SUBSTATION

- ( ~~B~~ ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}C \pm 0.2^{\circ}C$
- ( ✓ ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ( ✓ ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ( ✓ ) 4. Air blank completed.
- ( ✓ ) 5. Calibration check completed. Test results 0. 094 AC.
- ( ~~B~~ ) 6. Air blank completed.
- ( ✓ ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  #557

DPS Form Exh G-2 (Rev 05-01)

Historical Note

New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000

Location: EHRENBERG STATION

Serial Number: 80-001198

Core Version: 8105.48

01/23/2015

10:52:44

-----  
Standard Lot#: 08513100A1

Last Changed By: M. JOHNSON #531  
-----

QAS: R. ROSADO #557

LA PAZ CSO  
-----

Operator: R. ROSADO #557

LA PAZ CSO  
-----

Test	g/210L	Time
Air Blank	0.000	10:53:15
0.100 Cal Check	0.094	10:53:36
Air Blank	0.000	10:54:05

Preventative Maintenance  
Performed -- Timer Reset