

Department of Public Safety - Alcohol Testing

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST R. Rosado #557 AGENCY LA. PAZ COUNTY SO
DATE 01-03-15 09:51 TIME
INTOXILYZER SERIAL # 80-000934 DPS LOCATION LPCS0 JAIL

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ} C \pm 0.2^{\circ} C$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0.101 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE R. Rosado #557

DPS Form Exh G-2 (Rev 05-01)

Historical Note

New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

INTOXILYZER 8000

Location: LPCSO JAIL

Serial Number: 80-000934

Core Version: 8105.48

01/23/2015

09:51:17

Standard Lot#: 08513100A1

Last Changed By: M. JOHNSON #531

QAS: R. ROSADO #557

LA PAL CSO

Operator: R. ROSADO #557

LA PAL CSO

Test	g/210L	Time
Air Blank	0.000	09:51:53
0.100 Cal Check	0.101	09:52:14
Air Blank	0.000	09:52:43

Preventative Maintenance
Performed -- Timer Reset